



May 2018

To clean Hopkins® Series of Nylon or Cotton Latex Free Blood Pressure Sphygmomanometers:

Gauge/bulb/valve and tubing can be sprayed or wiped down with surface disinfectant solution such as Sanizide Plus, Cavicide or PDI. Spray or wipe versions of these or similar products are acceptable. It is suggested to NOT use a bleach based product to ensure colors are not degraded on the cuff.

Spray surface disinfectant solution liberally onto cuff and wipe down with sterile cloth over entire cuff surface. Wipe off excess solution with sterile cloth and allow cuff to air dry. Alternatively you can apply a wipe version of the disinfectant solution.

NOTE: Please adhere to disinfectant procedure for the manufacturer of your wipe/disinfectant. That procedure would also suffice for Hopkins® Series BP units. Dry/wait time as dictated by those disinfectant manufacturers is recommended.

CAUTION: Do not iron cuff. CAUTION: Do not heat or steam sterilize cuff.